**Parcus 5.5 – 6.5mm Punch/Tap  
with Handle**

**Important Product Information**  
Produktinformationsblatt  
Información Importante sobre el Producto  
Informations Importantes sur le Produit

**Directions for Use**  
Gebrauchsanleitung  
Instrucciones de uso  
Mode d'emploi

---

30387 Rev 4

[www.ParcusMedical.com](http://www.ParcusMedical.com)  
(941)755-7965

Mfd by:  
Parcus Medical, LLC  
6423 Parkland Dr.  
Sarasota, FL 34243

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## Parcus 5.5 – 6.5mm Punch/Tap with Handle

### 1. Indications:

The Parcus 5.5 – 6.5mm Punch/Tap with Handle is designed to prepare bone for the implantation of the Parcus 5.5mm and 6.5mm V-LoX™ PEEK CF and Twist PEEK Suture Anchors.

### 2. Warnings:

- A. This product is intended for use by or on the order of a physician.
- B. This product should not be used to implant any device other than the Parcus V-LoX PEEK CF or Twist PEEK Suture Anchors.
- C. Do not use this product if it is bent or there is visible damage to the tip or thread form.

### 3. Material:

This device is manufactured from a Stainless Steel Alloy shaft and an Aluminum handle. The materials used in the manufacture of this device that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

### 4. Cleaning:

- A. Immediate rinsing and cleaning after use with an enzymatic detergent will effectively remove and prevent drying of adherent blood, tissue, etc.
- B. Scrub device with a soft brush, paying special attention to areas where debris might accumulate. Always avoid any harsh materials that can scratch or mar the surface of the device.
- C. Rinse the device thoroughly with water following the cleaning process.

### 5. Sterilization:

This device is not provided sterile. The following table provides recommended minimum sterilization parameters that have been validated by Parcus Medical to provide a 10<sup>-6</sup> sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Gravity	132°C / 270°F	15 minutes	30 minutes <sup>1</sup>
Pre-vacuum	132°C / 270°F	4 minutes	

<sup>1</sup> Drying times vary according to load size and should be increased for larger loads.

### 6. Directions for use:

- A. Identify the site for anchor insertion and gently tap the Parcus Punch/Tap into the bone until the threads of the Punch/Tap can be engaged.
- B. Rotate the Punch/Tap clockwise into the bone until the top of the laser line is flush with the bone surface.
- C. Carefully remove the Punch/Tap by rotating the handle in a counter-clockwise direction until its threaded portion is clear of the bone.
- D. Insert a Parcus V-LoX PEEK CF or Twist PEEK Suture Anchor into the prepared hole.

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