
Parcus Traction Unit (PTU) Directions for Use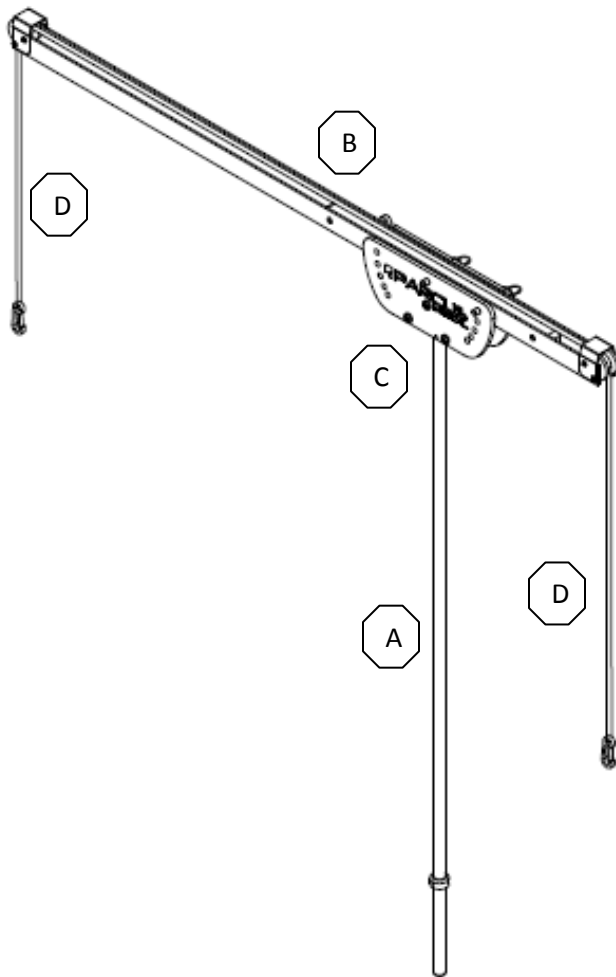

Indications:

The Parcus Traction Unit (PTU) is a fully adjustable medical support device which interfaces with the operating table and is used to assist in shoulder surgery by positioning the arm as indicated by the requirements of the procedure.

Warnings:

- This product is intended for use by or on the order of a physician.
- No component of this instrument is intended to be used in or left in the body.
- The PTU should not be used to apply more than 20 lbs (9 kg) of tension and should not be used for extended periods of time.

Material:

This product is manufactured from stainless steel, anodized aluminum alloys, plastics and nylon rope.

Sterilization and Cleaning:

This device is not provided sterile nor is it intended to be sterilized. It should be cleaned per facility standards using validated protocols with a damp cloth and mild detergent. If components not easily cleaned become grossly contaminated, they should be discarded and replaced.

Directions for use:

- 1) Securely attach the vertical stainless pole (A) to the operating table.
- 2) Place horizontal crossbar (B) (with rollers up, as shown in Figure 1) into housing (C) of vertical stainless pole.
- 3) Align center hole of housing with desired hole in crossbar. Place one push-button pin through anterior and posterior holes of both housing and crossbar.
- 4) Place second push-button pin through desired anterior and posterior holes on perimeter of housing and corresponding crossbar holes. Use of each successive perimeter housing hole raises/lowers end of crossbar.
- 5) Thread nylon rope (D) along length of crossbar and over rollers.
- 6) Attach the nylon rope to the loop on the Parcus Traction Kit that has already been placed over the patient's arm.
- 7) Apply the necessary counterweight to the opposite end of the nylon rope to provide the required tension, ensuring to not exceed the 20 lbs maximum.

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