Parcus Trapezium Removal Tool

Important Product Information

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Parcus Trapezium Removal Tool (English)

1. Indications:

The Parcus Trapezium Removal Tool is designed to aid in removal of the trapezium bone.

2. Warnings:

- A. Caution: Federal Law restricts this device to sale by or on the order of a physician.
- B. Do not use this product if it is bent or there is visible damage to the shaft or drive section.
- C. This device is not intended to be used as an implant.

3. Material:

This device is manufactured from a Stainless Steel Alloy and may include an aluminum alloy handle. The materials used in the manufacture of this device are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

4. Inspection:

- A. Inspect the device for damage at all stages of handling.
- B. If damage is detected, consult the manufacturer for guidance.

5. Cleaning:

- A. Immediate rinsing and cleaning after use with an enzymatic detergent will effectively remove and prevent drying of adherent blood, tissue, etc.
- B. Scrub device with a soft brush, paying special attention to areas where debris might accumulate. Always avoid any harsh materials that can scratch or mar the surface of the device.
- C. Rinse the device thoroughly with water following the cleaning process.

6. Sterilization:

- A. This device is not provided sterile. It should be cleaned and sterilized using validated protocols before each use.
- B. The following table provides recommended minimum sterilization parameters that have been validated by Parcus Medical to provide a 10^{-6} sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Pre-Vacuum	132°C / 270°F	4 Minutes	30 Minutes ¹

¹Drying times vary according to load size and should be increased for larger loads.

7. Instructions for use:

- A. Place the tip of the Trapezium Removal Tool on the surface of the bone. Dense bone may require that a pilot hole be created to more easily engage the threads in the bone.
- B. Thread the Trapezium Removal Tool into the trapezium so that all of its threads are fully inserted and flush with the surrounding bone.
- C. Use the Trapezium Removal Tool as a lever to create space to use suitable instruments to free the trapezium from its attachments to the surrounding bones.
- D. When free from the surrounding bone, avulse and remove the trapezium.

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